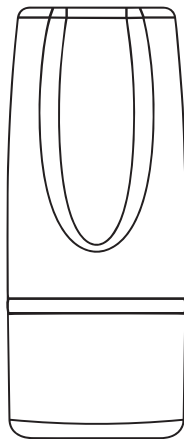
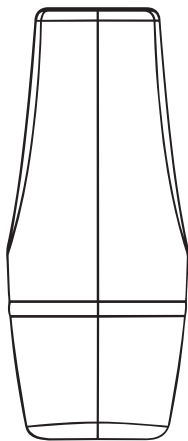
manigrip

# WINI

## INSTRUCTIONS FOR USE

*Grip enlarger (EN) / Grebsforstørrer (DK) / Grepphjälpmedel (SV) /  
Grip forstørrer (NO) / Otteen laajentaja (FI) / Gripstækkari (IS) /  
Agrandisseur de poignée (FR) / Greepvergroter (NL) /  
Griffstärke-Vergrößerungsgerät (DE)*

---

CE

MD

# CONTENT

Instructions for use (EN)	<b>4</b>
Brugermanual (DK)	<b>14</b>
Bruksanvisning (SV)	<b>24</b>
Bruksanvisning (NO)	<b>34</b>
Käyttöohjeet (FI)	<b>44</b>
Notkunarleiðbeiningar (IS)	<b>54</b>
Mode d'Emploi (FR)	<b>64</b>
Gebruiksaanwijzing (NL)	<b>74</b>
Gebrauchsanweisung (DE)	<b>84</b>

# INSTRUCTIONS FOR USE

English

## IMPORTANT

---

This User Manual contains important safety instructions and information regarding the use of Wini.

This symbol indicates important information related to safety. Follow these instructions carefully.

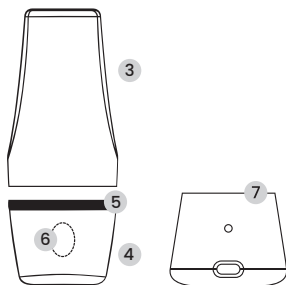
Read User Manual before use!  
Wini is simple to use, but please read through the User Manual to learn how to use the device correctly.

*Visit [www.manigrip.dk](http://www.manigrip.dk) to ensure you have the latest User Manual.*

## Handle

## Cover

## Parts

1. Button
2. Charging pins
3. Cover piece 1
4. Cover piece 2
5. Silicone ring
6. Button
7. Docking station

**CE declaration**

The product, and its related accessories, described in this user manual, is in compliance with the regulation (EU) 2017/745 of 25th October 2022 – as a medical device, risk class I.

Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Wini is a versatile aid device that can be used to enlarge the gripping surface of small everyday objects such as cutlery, paint brushes, crochet hooks, pens, toothbrushes etc. The flexible closing mechanism inside the device allows you to try out lots of different objects - be creative!

Wini is co-created with arthritis sufferers and occupational therapists. Insights from more than 120 patients has shown the desire to have aid devices that are flexible, functional and aesthetic, all of which has been a focus point in designing Wini. Wini is furthermore named after the first patient, who shared their insights with Manigrip.

Visit [manigrip.dk](http://manigrip.dk) for more tips and tricks.

**Intended purpose:**

Wini is a hand-held grip aid device for hand instrument manipulation used to facilitate the gripping of articles of daily use for people with restricted mobility of the hands.

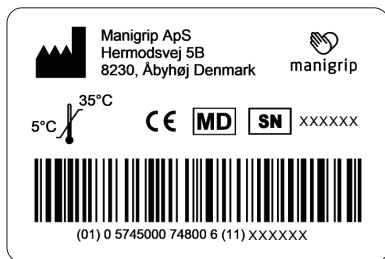
**Intended target group and environment:**

Wini can be used by people with restricted mobility of the hands. The device can be useful for people with; arthritis, multiple sclerosis, nerve damage, spinal cord injuries, elderly or other illnesses that affect the mobility of the hands. The device can be used in the home, on the go and in a professional environment.

# PRODUCT IDENTIFICATION

EN

The Product label's barcode contain serial number and production date:

## SYMBOLS

CE-marked

Manufacturer

Do not dispose in  
general waste

Serial number

Medical Device

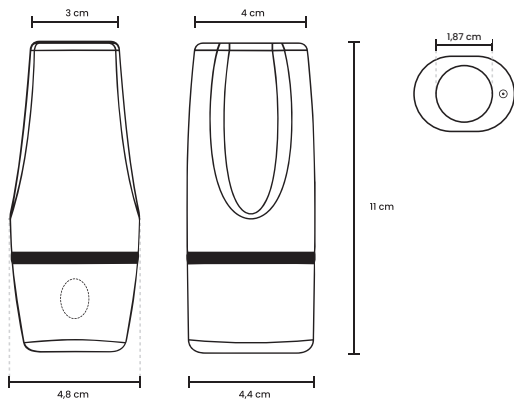
Read the user manual

# TECHNICAL DATA

EN

Materials:	Plastic, silicone
Battery capacity:	150 mAh
Closing/opening cycle:	Closing/opening cycle depends on the size of the objects and varies from 2-5 seconds.
Charging time:	60 min.
Weight:	114 g. pr. handle
Ingress protection (IP):	Protected from immersion between 15 centimetres and 1 metre in depth (IP47).
Temperature for operating:	-10 ~ + 60°C
Temperatur for storing:	-45 ~ 60°C, 60% RH: 1 month - 25 ~ 45°C, 60% RH: 3 months -20 ~ 25°C, 60% RH: 12 months Stored 50% charged.

# DIMENSIONS

## MISUSE/WARNINGS

EN

Don't press the button, without an object inserted into the handle. It will decrease the expected lifetime of Wini.

Do not clean the handle in the dishwasher, only the covers.

Don't charge the handle while wet. Make sure it is completely dry before inserting the handle into the docking station.

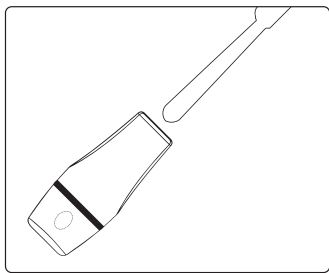
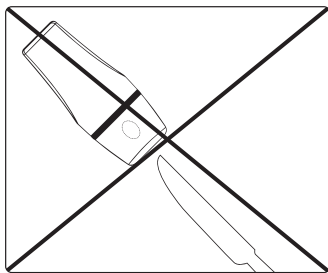
Don't insert bodily parts in the handle.

Do not pull on the object when the object is held in the grip. This can damage the grip. Instead, press the button to release the object.

Don't put sharp objects through the handle, it can damage the function. Instead insert objects such as knives, handle first.



## Before use


It is easy to get started, but make sure to read through the instructions, clean the handle and charge the handle completely (see page 20-21).

## Function

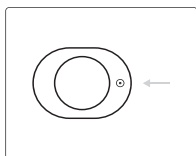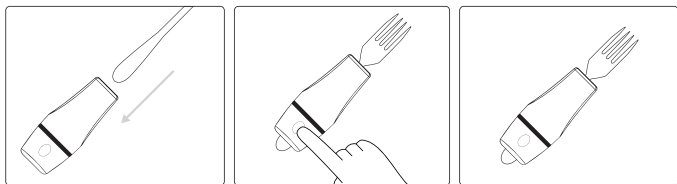Insert the object you want to get a larger surface of into the handle, and press the button. The button will activate the closing mechanism and automatically hold on to the object. Press the button again and the handle will let go of the object.

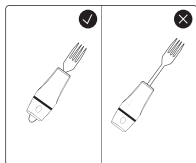**NOTE:** The object may rotate slightly. Use the round mark as an indicator, to learn how to position the object to achieve the desired angle.


**NOTE:** To get the most stable hold on the object, place the object so it sticks out as evenly as possible in both ends as shown in the illustration.

# CHARGING AND INDICATIONS

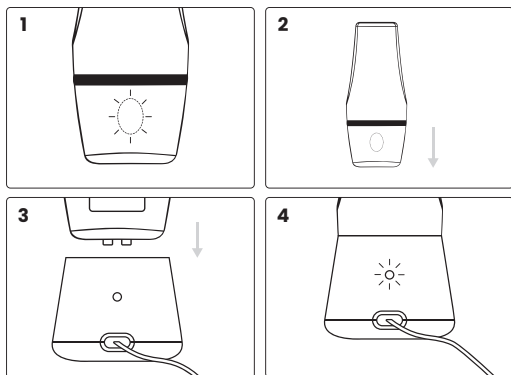EN

Wini is low on battery when you see a flashing red light, while pressing the button.


To start charging, remove cover piece 2 and place the handle in the docking station. Make sure the docking station is plugged into a USB-C cable. You will see a green flashing light, that indicates the handle is charging. When the handle is fully charged the green light will switch to constant green.


## Light indications

Flashing red	10% battery
Flashing green (inserted in docking station)	Charging
Constant green (inserted in docking station)	100% battery
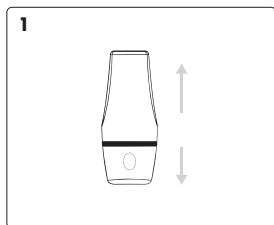
## Cleaning of cover

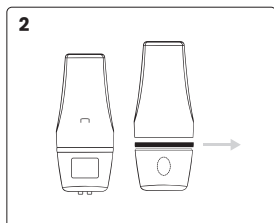
Remove the cover from the handle, by pulling apart the two cover pieces. If it is hard to remove, give it a light twist.

Remove the silicone ring from the cover.

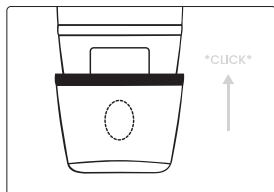Clean the cover in either the dishwasher or by hand.

Put the silicone ring back on the cover after cleaning, and click the cover back on. **NOTE:** Make sure the buttons from both the handle and cover overlap.

## Cleaning of handle

Clean the outside with a damp cloth.

The inside can be cleaned with a damp cloth, pipe cleanser or anything soft that fits in the opening. Do not clean the silicone tube with harsh detergents. Only use water or very mild soap to clean the inner tube.

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